

Instruction for Use

Product: Light Handle Cover

The Following information should be read before using this device.

Description:

The Disposable Light Handle Cover is designed for provided for a sterile field for surgeon grip the OR light during procedure. It is designed for single use and not for re-sterilization.

Preparation:

1. Remove the sleeve containing the Disposable Light Handle Cover from the peel pouch package.
2. Then assembly into OR light handle directly.
3. Dispose Disposable Light Handle Cover.












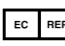


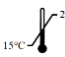

Cautions/Warnings:

This device was designed, tested and manufactured for single patient use only. Reuse or reprocessing of this device may lead to its failure and subsequent patient injury. Reprocessing and/or re-sterilization of this device may create the risk of contamination and patient infection. Do not reuse, reprocess or re-sterilize this device.


Contents sterile unless enclosed package has been opened or damaged. Store at 15°C~25°C.

Dispose used product following local state or federal guidelines.

Federal (U.S.A) law restricts this device to sale, distribution, and use by, or on the order of a physician.

 Caution	 Sterilized using Ethylene Oxide	 Consult instructions for use
 Do not use if package is damaged	 Batch code	 Do not reuse
 Keep away from sunlight	 Date of manufacture	 Keep dry
 Catalogue number	 Manufacturer	 Authorised representative in the European community
 Do not re-sterilize	 Use by	 Temperature limitation: 15°C~25°C
 Notified body: DNV GL Presafe AS, with CE2460		

If further information is required, please contact:

 Advanced Medical Design Co., Ltd.
 4F~5F, No29, Wuquan 5th Rd., Wugu Dist.,
 New Taipei City, 24888 Taiwan R.O.C.
 TEL: 886-2-22902627 FAX: 886-2-22988501
 Email: sales@amdintl.com.tw

 Lotus NL B.V.
 Koningin Julianaplein 10, 1e Verd, 2595AA,
 The Hague, Netherlands.
 Tel: +31645171879
 Email: peter@lotusnl.com