

Instruction for Use

Product: Discard Pad

The Following information should be read before using this device.

Description:

The Discard Pad is designed for dispose surgical blade and needle after procedure.

It is available for discard pad with blister tray or light adhesion blister tray for use in Operation rooms.

Preparation:

1. Remove the sleeve containing the Discard Pad from the peel pouch package. Tear the film liner off, put the disposable sharp blade or needle into the glue area, after procedure, then fold it completely.
2. Please press and hold the (cling/ transparent) film between the release paper and the foam, and then tear the release paper and the foam. Failure to press and hold the (cling/ transparent) film may cause the damage to the foam.
3. Dispose discard pad. The used discarded in the compliance with government regulations for infectious waste disposal.

Cautions/Warnings:

This device was designed, tested and manufactured for single patient use only. Reuse or reprocessing of this device may lead to its failure and subsequent patient injury. Reprocessing and/or re-sterilization of this device may create the risk of contamination and patient infection. Do not reuse, reprocess or re-sterilize this device.

Contents sterile unless enclosed package has been opened or damaged. Store at 15°C~25°C.

Dispose used product following local state or federal guidelines.

Federal (U.S.A) law restricts this device to sale, distribution, and use by, or on the order of a physician.



Caution



Sterilized using Ethylene Oxide



Consult instructions for use



Do not use if package is damaged



Batch code



Do not reuse



Keep away from sunlight



Date of manufacture



Keep dry



Catalogue number



Manufacturer



Authorized representative in the European community



Do not re-sterilize



Use by



Temperature limitation: 15°C~25°C



Notified body: DNV GL Presafe AS, with CE2460

If further information is required, please contact:



Advanced Medical Design Co., Ltd.
 4F~5F, No29, Wuquan 5th Rd., Wugu Dist.,
 New Taipei City, 24888 Taiwan R.O.C.
 TEL: 886-2-22902627 FAX: 886-2-22988501
 Email: sales@amdintl.com.tw



Lotus NL B.V.
 Koningin Julianaplein 10, 1e Verd, 2595AA,
 The Hague, Netherlands.
 Tel: +31645171879
 Email: peter@lotusnl.com