

Instruction for Use

Product: Monopolar and Bipolar Connecting Cable

The Following information should be read before using this device.

Description:

The Disposable Monopolar & Bipolar Cable is connected with monopolar or bipolar electro-surgical capacity, as Laparoscopic Instrumentation with standard EU adaptor.

Preparation:

1. Open the pouch and take the cable out, connect adaptor into plug of instrument with monopolar or bipolar electro-surgical capacity.
2. Forward plug to the scout nurse/theatre technician to connect plug into generator.
3. To operate the monopolar or bipolar electro-surgical feature, please follow instructions provide by electro-surgical generator manufacturer.

Cautions/Warnings:

1. This device was designed, tested and manufactured for single patient use only. Reuse or reprocessing of this device may lead to its failure and subsequent patient injury. Reprocessing and/or re-sterilization of this device may create the risk of contamination and patient infection. Do not reuse, reprocess or re-sterilize this device.
2. Contents sterile unless enclosed package has been opened or damaged. Store at 15°C~25°C.
3. Federal (U.S.A) law restricts this device to sale, distribution, and use by, or on the order of a physician.
4. Endoscopic procedures should be performed only by physicians with adequate training and knowledge of these procedures. In addition, medical literature should be consulted regarding techniques, hazards, contraindications, and complications prior to the performance of these procedures.
5. Before endoscopic instruments and accessories from different manufacture are utilize together in a procedure, verify compatibility and ensure that electrical isolation of grounding is not compromised.
6. A thorough understanding of techniques and principals involved in electro-surgical procedures is necessary to avoid burn or shock hazard to patient and/or operator.
7. This instruments should be used with U.L recognized bipolar electro-surgical generators, with capability of being activated with a foot control switch.
8. Keep the voltage power as low as possible to achieve the desired end effect, refer to use instructions provided by the electro-surgical generator to ensure that all safety precautions are followed.
9. Dispose used product following local state or federal guidelines.



Caution



Sterilized using Ethylene Oxide



Consult instructions for use



Do not use if package is damaged



Batch code



Do not reuse



Keep away from sunlight



Date of manufacture



Keep dry



Do not re-sterilize



Use by



Authorised representative in the European community



Catalogue number



Manufacturer



Temperature limitation: 15°C~25°C



Notified body: DNV Product Assurance AS, with CE2460



Advanced Medical Design CO., LTD.

4F~5F, No29, Wuquan 5th Rd., Wugu Dist., New Taipei city, 248 Taiwan
TEL: 886-2-22902627 FAX: 886-2-22988501

If further information is required, please contact:



Advanced Medical Design Co., Ltd.
4F~5F, No29, Wuquan 5th Rd., Wugu Dist.,
New Taipei City, 24888 Taiwan R.O.C.
TEL: 886-2-22902627 FAX: 886-2-22988501
Email: sales@amdintl.com.tw



OBELIS S.A
Bd. Général Wahis, 53, 1030 Brussels, Belgium
TEL: +3227325954
FAX: +3227326003
E-mail: mail@obelis.net