

Instruction for Use

Product: Camera Sleeves

The Following information should be read before using this device.

Description:

The Camera Sleeve is a sterile, single use, disposable drape made of tough, thin polyethylene, designed for surgical procedures. The disposable Camera Sleeve is the quickest and simplest way to create a sterile field around your video system during operative laparoscopic procedure.


















Preparation:

1. Open peel pouch and aseptically drop Camera Sleeve onto tray.
2. Insert the camera head through sleeve and remove the paper from the front. Connect with sterile telescope.
3. Gently pull the blue sealing tape off and slightly away from ring and wrap the tape, sealing first to the telescope and then to sleeve. Use additional blue tape if necessary.
4. With the scrub nurse holding the telescope with one hand, push ring all the way down the cable, and then the roving staff pulls the ring further down the cable, being careful not to touch the surface of the drape.
5. If desired, scrub nurse may remove the second sealing tape from the ring to use later for stabilizing the sleeve to the cable near the telescope to stop sleeve from bunching up during use.



Cautions/Warnings:

- The disposable Camera Sleeve is designed for single use only, and is not to be re-used or re-sterilized.
- Contents sterile unless enclosed package has been opened or damaged. Store at 15°C~25°C.
- Dispose used product following local state or federal guidelines.
- Federal (U.S.A) law restricts this device to sale, distribution, and use by, or on the order of a physician.

 Caution	 Sterilized using Ethylene Oxide	 Consult instructions for use
 Do not use if package is damaged	 Batch code	 Do not reuse
 Keep away from sunlight	 Date of manufacture	 Keep dry
 Catalogue number	 Manufacturer	 Temperature limitation: 15°C~25°C
 Do not re-sterilize	 Use by	 Medical device
 Single sterile barrier system	 Single sterile barrier system with protective packaging outside	

If further information is required, please contact:



Advanced Medical Design Co., Ltd.
 4F~5F, No29, Wuquan 5th Rd., Wugu Dist.,
 New Taipei City, 24888 Taiwan R.O.C.
TEL: 886-2-22902627
FAX: 886-2-22988501
<https://www.amdintl.com.tw/>
 Email: sales@amdintl.com.tw