

## Instruction for Use

### Product: Sharp Box

The Following information should be read before using this device.

#### Description:

The Sharp management box is designed for dispose surgical blade and needle after procedure. It is available for various size and foam, magnetic pad for use in Operation Theater.

#### Preparation:

1. Remove the sleeve containing the Sharp management box from the peel pouch package. Open the box and using the adhesion type to fixed in procedure area.
2. Put the dispose sharp blade or needle into the glue area, after procedure, then closed it completely.
3. Dispose Sharp management box.












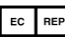




#### Cautions/Warnings:

This device was designed, tested and manufactured for single patient use only. Reuse or reprocessing of this device may lead to its failure and subsequent patient injury. Reprocessing and/or re-sterilization of this device may create the risk of contamination and patient infection. Do not reuse, reprocess or re-sterilize this device.


Contents sterile unless enclosed package has been opened or damaged. Store at 15°C~25°C.

Dispose used product following local state or federal guidelines.

Federal (U.S.A) law restricts this device to sale, distribution, and use by, or on the order of a physician.

 <b>Caution</b>	 <b>Sterilized using Ethylene Oxide</b>	 <b>Consult instructions for use</b>
 <b>Do not use if package is damaged</b>	 <b>Batch code</b>	 <b>Do not reuse</b>
 <b>Keep away from sunlight</b>	 <b>Date of manufacture</b>	 <b>Keep dry</b>
 <b>Catalogue number</b>	 <b>Manufacturer</b>	 <b>Authorised representative in the European community</b>
 <b>Do not re-sterilize</b>	 <b>Use by</b>	 <b>Temperature limitation: 15°C~25°C</b>
 <b>Notified body: DNV GL Presafe AS, with CE2460</b>		

#### If further information is required, please contact:

 Advanced Medical Design Co., Ltd.  
 4F~5F, No29, Wuquan 5th Rd., Wugu Dist.,  
 New Taipei City, 24888 Taiwan R.O.C.  
 TEL: 886-2-22902627 FAX: 886-2-22988501  
 Email: sales@amdintl.com.tw

 **EC REP**

Lotus NL B.V.  
 Koningin Julianaplein 10, 1e Verd, 2595AA,  
 The Hague, Netherlands.  
 Tel: +31645171879  
 Email: peter@lotusnl.com