

Instruction for Use

Product: Magnetic Instrument Mat

The Following information should be read before using this device.

Description:

The Disposable Magnetic Instrument Mat is designed for transfer sharp blade or instrument during procedure. It is designed for single use and not for re-sterilization.

Preparation:

1. Remove the sleeve containing the Disposable Magnetic Instrument Mat from the peel pouch package.
2. Scrub nurse assembly or put the dispose sharp blade or instrument into the Magnetic Mat, then forward to surgeon for procedure demand.
3. Dispose Magnetic Instrument Mat.

Cautions/Warnings:

This device was designed, tested and manufactured for single patient use only. Reuse or reprocessing of this device may lead to its failure and subsequent patient injury. Reprocessing and/or re-sterilization of this device may create the risk of contamination and patient infection. Do not reuse, reprocess or re-sterilize this device.

Contents sterile unless enclosed package has been opened or damaged. Store in room temperature.

Dispose used product following local state or federal guidelines.

Federal (U.S.A) law restricts this device to sale, distribution, and use by, or on the order of a physician.



Caution



Sterilized using Ethylene Oxide



Consult instructions for use



Do not use if package is damaged



Batch code



Do not reuse



Keep away from sunlight



Date of manufacture



Keep dry



Catalogue number



Manufacturer



Authorised representative in the
European community



Do not re-sterilize




Use by



Notified body: DNV GL Business
Assurance Norway, with CE0434

If further information is required, please contact:

Manufacturer by:

 ADVANCED MEDICAL DESIGN CO., LTD.
 4F~5F, No29, Wuquan 5th Rd., Wugu Dist.,
 New Taipei City, 24888 Taiwan R.O.C.
 TEL: 886-2-22902627 FAX: 886-2-22988501
 Email: sales@amdintl.com.tw



Lotus Global Co., Ltd.
 1 Four Season Terrace, West Drayton,
 Middlesex.London, UB7 9GG. U.K.
 Tel : 44-20-75868010 Fax: 44-20-79006187

Instruction for Use

Product: Magnetic Instrument Mat

The Following information should be read before using this device.

Description:

The Magnetic Instrument Mat is designed for transfer sharp blade or instrument during procedure. Provides a magnetized non-slip surface for the safe placement. Need to be sterilized before using.

Preparation:

1. Follow the hospital sterilization procedure to sterilize the Magnetic Instrument Mat.
2. The circulating nurse transfers the Magnetic Instrument Mat to scrub nurse.
3. Scrub nurse put the disposed sharp blade or instrument on the Magnetic Instrument Mat, then forward to surgeon for procedure demand.
4. After use, follow the hospital cleaning procedure to clean the Magnetic Instrument Mat.

Cautions/Warnings:

- Magnetic Instrument Mat must be sterilized prior to use and before entering the sterile field.
- The device is reusable and must follow the hospital cleaning and sterilizing procedures.
- Dispose used product in accordance with hospital, administrative and/or local government policy.
- Federal (U.S.A) law restricts this device to sale by or on the order of a physician.
- Do not use it again if it is found to be damaged.



Caution



Medical device



Consult instructions for use



Do not use if package is damaged



Batch code



Use by



Keep away from sunlight



Date of manufacture



Keep dry



Catalogue number



Manufacturer

If further information is required, please contact:



Advanced Medical Design Co., Ltd.
4F~5F, No29, Wuquan 5th Rd., Wugu Dist.,
New Taipei City, 24888 Taiwan R.O.C.
TEL: 886-2-22902627
FAX: 886-2-22988501
<https://www.amdintl.com.tw/>
Email: sales@amdintl.com.tw