

Instruction for Use

Product: Smoker Filter with Tubing Set

The Following information should be read before using this device.

Intended Use:

To be used in conjunction with MIS procedure kit for intra-abdominal Laparoscopic surgery and procedure.

Instruction for Use:

1. Tear off cover to access sterile contents. Place contents on sterile field.
2. Connect the luer connector to the surgical trocar cannula.
3. Connect the suction evacuation machine with the connector of filter for active smoke evacuation, or not connect for passive smoke evacuation.
4. Adjust or switch the flow rate with the roller clamp.

Cautions/Warnings:

Completely inspect package prior to use. Do not use if the sterile packaging is opened, damaged or wet. Re-sterility might be compromised.

Dispose used product following local state or federal guidelines.

Contents sterile unless enclosed package has been opened or damaged. Store at 15°C~25°C.

Federal (U.S.A) law restricts this device to sale, distribution, and use by, or on the order of a physician.

In high-humidity scenario, the filter may cause blockage due to moisture inhalation. Please replace the product immediately.



Caution



Sterilized using Ethylene Oxide



Consult instructions for use



Do not use if package is damaged



Batch code



Do not reuse



Keep away from sunlight



Date of manufacture



Keep dry



Do not re-sterilize



Use by



Temperature limitation: 15°C~25°C



Catalogue number



Manufacturer



Authorised representative in the European community



Notified body: DNV Product Assurance AS, with CE2460

If further information is required, please contact:



Advanced Medical Design Co., Ltd.
 4F~5F, No29, Wuquan 5th Rd., Wugu Dist.,
 New Taipei City, 24888 Taiwan R.O.C.
 TEL: 886-2-22902627 FAX: 886-2-22988501
 Email: sales@amdintl.com.tw



OBELIS S.A
 Bd. Général Wahis, 53, 1030 Brussels, Belgium
 TEL: +3227325954
 FAX: +3227326003
 E-mail: mail@obelis.net