

Instruction for Use

Product: Insufflation Filter with Tubing Set

The Following information should be read before using this device.

Intended Use:

To be used in conjunction with MIS procedure kit for intra-abdominal Laparoscopic surgery and procedure.

Note: Single use only, Sterile (Unless package is damaged or opened). All contents are ETO sterilized.

Instruction for Use:

1. Tear off cover to access sterile contents.
2. Place contents on sterile field.
3. Give the connection fitting closest to filter to the nurse for connection to output insufflation gas port.
4. For barbed output, simply cut off fitting and insert over barbed fitting.
5. Connect the opposite Luer connector in sterile field to Veress Needle or gas port of Surgical Trocar Cannula.
6. Proceed with insufflation.

Cautions/Warnings:

This device was designed, tested and manufactured for single patient use only. Reuse or reprocessing of this device may lead to its failure and subsequent patient injury. Reprocessing and/or re-sterilization of this device may create the risk of contamination and patient infection. Do not reuse, reprocess or re-sterilize this device.

Contents sterile unless enclosed package has been opened or damaged. Store at 15°C~25°C.

Dispose used product following local state or federal guidelines.

Federal (U.S.A) law restricts this device to sale, distribution, and use by, or on the order of a physician.



Caution



Sterilized using Ethylene Oxide



Consult instructions for use



Do not use if package is damaged



Batch code



Do not reuse



Keep away from sunlight



Date of manufacture



Keep dry



Do not re-sterilize



Use by



Authorised representative in the European community



Catalogue number



Manufacturer



Temperature limitation: 15°C~25°C



Notified body: DNV Product Assurance AS, with CE2460

If further information is required, please contact:



Advanced Medical Design Co., Ltd.
 4F~5F, No29, Wuquan 5th Rd., Wugu Dist.,
 New Taipei City, 24888 Taiwan R.O.C.
 TEL: 886-2-22902627 FAX: 886-2-22988501
 Email: sales@amdintl.com.tw



OBELIS S.A
 Bd. Général Wahis, 53, 1030 Brussels, Belgium
 TEL: +3227325954
 FAX: +3227326003
 E-mail: mail@obelis.net