

Instruction for Use

Product: Trocar with Cannula

The Following information should be read before using this device.

Description:

The Surgical Trocar set is intend to use for Laparoscopic procedure, it can be provide a port for instrument into abdominal, product are available for Disposable model only, has several length for difference patient and procedure request.

Contraindications:

This device is not intended for use when endoscopic techniques are contraindicated.

Preparation:

1. Remove the product from the package tray.
2. To assembly with obturator and cannula and hold straight and insert into abdominal, the inner shaft will engage shield and protect to damage inside organ.
3. To connect Insufflation generator with luer lock adaptor into needle or Trocar's gas port for insufflation gas into patient's abdominal.
4. Please follow instructions provided by Insufflation pump manufacturer.

Cautions/Warnings:

This device was designed, tested and manufactured for single patient use only. Reuse or reprocessing of this device may lead to its failure and subsequent patient injury. Reprocessing and/or re-sterilization of this device may create the risk of contamination and patient infection. Do not reuse, reprocess or re-sterilize this device.

Contents sterile unless enclosed package has been opened or damaged. Store at 15°C~25°C.

Dispose used product following local state or federal guidelines.

Federal (U.S.A) law restricts this device to sale, distribution, and use by, or on the order of a physician.



Caution



Sterilized using Ethylene Oxide



Consult instructions for use



Do not use if package is damaged



Batch code



Do not reuse



Keep away from sunlight



Date of manufacture



Keep dry



Catalogue number



Manufacturer



Authorised representative in the European community



Do not re-sterilize



Use by



Temperature limitation: 15°C~25°C



Notified body: DNV Product Assurance AS, with CE2460

If further information is required, please contact:



Advanced Medical Design Co., Ltd.
 4F~5F, No29, Wuquan 5th Rd., Wugu Dist.,
 New Taipei City, 24888 Taiwan R.O.C.
 TEL: 886-2-22902627 FAX: 886-2-22988501
 Email: sales@amdintl.com.tw



OBELIS S.A
 Bd. Général Wahis, 53, 1030 Brussels, Belgium
 TEL: +3227325954
 FAX: +3227326003
 E-mail: mail@obelis.net