

## Instruction for Use

### Product: Veress Needle

The Following information should be read before using this device.

#### Description:

The Veress Needle is intended for percutaneous insertion into the peritoneal cavity for the purpose of insufflation with carbon dioxide to establish pneumoperitoneum prior to the placement of trocars during laparoscopic procedures.

#### Contraindications:

This device is not intended for use when endoscopic techniques are contraindicated.

#### Preparation:

1. Remove the product from the package tray.
2. To hold straight and insert into abdominal, the inner shaft will engage shield and protect to damage internal organ
3. To connect Insufflation generator with luer lock adaptor into needle or Trocar's gas port for insufflation gas into patient's abdominal.
4. Please follow instructions provided by Insufflation pump manufacturer.

#### Cautions/Warnings:

This device was designed, tested and manufactured for single patient use only. Reuse or reprocessing of this device may lead to its failure and subsequent patient injury. Reprocessing and/or re-sterilization of this device may create the risk of contamination and patient infection. Do not reuse, reprocess or re-sterilize this device.

Contents sterile unless enclosed package has been opened or damaged. Store at 15°C~25°C.

Dispose used product following local state or federal guidelines.

Federal (U.S.A) law restricts this device to sale, distribution, and use by, or on the order of a physician.



**Caution**



**Sterilized using Ethylene Oxide**



**Consult instructions for use**



**Do not use if package is damaged**



**Batch code**



**Do not reuse**



**Keep away from sunlight**



**Date of manufacture**



**Keep dry**



**Catalogue number**



**Manufacturer**



**Authorised representative in the European community**



**Do not re-sterilize**



**Use by**



**Temperature limitation: 15°C~25°C**



**Notified body: DNV Product Assurance AS, with CE2460**

#### If further information is required, please contact:



Advanced Medical Design Co., Ltd.  
 4F~5F, No29, Wuquan 5th Rd., Wugu Dist.,  
 New Taipei City, 24888 Taiwan R.O.C.  
 TEL: 886-2-22902627 FAX: 886-2-22988501  
 Email: sales@amdintl.com.tw



OBELIS S.A  
 Bd. Général Wahis, 53, 1030 Brussels, Belgium  
 TEL: +3227325954  
 FAX: +3227326003  
 E-mail: mail@obelis.net