

Instruction for Use

Product: Sharp Box

The Following information should be read before using this device.

Description:

The Sharp management box is designed for dispose surgical blade and needle after procedure. It is available for various size and foam, magnetic pad for use in Operation Theater.

Preparation:

1. Remove the sleeve containing the Sharp management box from the peel pouch package. Open the box and using the adhesion type to fixed in procedure area.
2. Put the dispose sharp blade or needle into the glue area, after procedure, then closed it completely.
3. Dispose Sharp management box.












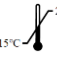


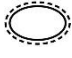


Cautions/Warnings:

This device was designed, tested and manufactured for single patient use only. Reuse or reprocessing of this device may lead to its failure and subsequent patient injury. Reprocessing and/or re-sterilization of this device may create the risk of contamination and patient infection. Do not reuse, reprocess or re-sterilize this device.

Contents sterile unless enclosed package has been opened or damaged. Store at 15°C~25°C.

Dispose used product following local state or federal guidelines.

Federal (U.S.A) law restricts this device to sale, distribution, and use by, or on the order of a physician.

 Caution	 Sterilized using Ethylene Oxide	 Consult instructions for use
 Do not use if package is damaged	 Batch code	 Do not reuse
 Keep away from sunlight	 Date of manufacture	 Keep dry
 Catalogue number	 Manufacturer	 Temperature limitation: 15°C~25°C
 Do not re-sterilize	 Use by	 Single sterile barrier system with protective packaging outside
 Medical device	 Single sterile barrier system	

If further information is required, please contact:



Advanced Medical Design Co., Ltd.
 4F~5F, No29, Wuquan 5th Rd., Wugu Dist.,
 New Taipei City, 24888 Taiwan R.O.C.
 TEL: 886-2-22902627
 FAX: 886-2-22988501
<https://www.amdintl.com.tw/>
 Email: sales@amdintl.com.tw